
Installation checkout procedure

Perform the following installation checkout procedures for the patient monitoring system. If any peripheral devices are connected to the patient monitor, perform the applicable tests.

Interface devices

Display

1. Verify the following are working properly:
 - ◆ alarm light
 - ◆ sounds
 - ◆ USB hub
 - ◆ keypad
 - ◆ touch
2. Verify that all text on the display screen is readable.
3. Verify that all images on the display screen are clear.

Touchscreen

Verify that touching a parameter window displays the screen information for that parameter.

Keypad and remote

1. Press the **Data & Pages** key and select the *Admit/Discharge* or *Start/End Case* by using the **TRIM KNOB**.
2. Verify that the **TRIM KNOB** can be used to navigate and select the *Admit/Discharge* or *Start/End Case* button.

Mouse

1. For a mouse that is connected to a USB port, move the mouse cursor over to **Data & Pages** and select *Admit/Discharge* or *Start/End Case*.
2. Verify that the cursor moves and a new window is displayed.

Keyboard

1. Make sure that the language of the keyboard and clinical application matches.
2. Navigate to *Edit Name & Number* window by selecting *Data & Pages > Admit/Discharge* or *Start/End Case > Edit Name & Number*.
3. Select the *Last Name* field and type all characters on the keyboard.
4. Verify that typed keys are correctly displayed.