
Assigning an Equipment Label

➤ **To assign an equipment label to a device:**

- 1 Select **All Controls** -> **Label Assignment**.
- 2 Enter password (tele)
- 3 Insert battery power into the MX40 and if attached, disconnect the patient cable.
- 4 Select **Refresh**.
- 5 Confirm the connection to the wireless network as follows:



- 6 Select the MAC address of the replacement device from the **New Devices** list. If the address does not appear, remove battery power and re-insert. Select **Refresh**.
Note — The MAC address appears on the rear label of the MX40.
- 7 Select the desired equipment label from the **Equipment Label** list.
- 8 Select **Assign Label** to initiate programming of the equipment label and RF Access Code into the MX40.
- 9 When prompted, press **Confirm** on the MX40 to accept the assignment. The confirmation must occur within 30 seconds of the prompt.
- 10 On the MX40, wait for the New_Device label to change to the selected equipment label.
- 11 Confirm the label assignment by viewing the waveform in the Patient Sector at the Information Center.