## **PHILIPS** Perform Label Assignment – iX/MX40

| 15bit - pusgamodalitydatacms58875974j9sa.prs.healthcare.philips.com - Remote Desktop Connection   |  |             |               |      |         | _ <b>_</b> ×  |
|---|--|-------------|---------------|------|---------|---------------|
| System Configuration - [SERVERIX1: locally connected] - [As   | sign Devices to Equipment              | Labels]     |               |      |         | _ 8 × ^       |
| 🖳 View Help   |  |             |               |      |         | PHILIPS - # × |
| Configure Jools Validate Archive Stop Services  |  |             |               |      |         |               |
| 2 PCMS  | Add Equipment Label                    | s Refresh   |               |      |         |               |
| Network Scan  | Zone: CCU - IXA                        |             |               |      |         |               |
| Language  |  |             |               |      |         |               |
| E- Clinical Settings  | Mac Address Serial Number Type Address |             |               |      |         |               |
| Display Management 3  | 123456789                              | A           | i)pc          | 4 (  | MX40-1  | -5- Assign    |
| Patient Management  |  |             |               |      | MIX40-2 |               |
| Local Surveillance  |  |             |               |      | MX40-3  | _             |
| Colors  |  |             |               |      | MX40-5  | =             |
| Global Settings   |  |             |               |      | MX40-6  |               |
| Telemetry Profiles  |  |             |               |      | -       | _             |
| Alarm Notification  |  |             |               |      | -       |               |
| Reports   |  |             |               |      |         |               |
| - White Recording   |  |             |               |      |         |               |
| ECG 12 Lead   | Assigned Devices:                      |             | 1             |      |         |               |
|   | Name                                   | Mac Address | Serial Number | Туре |         | Unassign      |
| <ol> <li>System Configuration</li> <li>Double click on label assignment</li> <li>MX40 powered on with NO leadset, select MAC address</li> <li>Select label to be assigned</li> <li>Select "Assign"</li> </ol> |  |             |               |      |         |               |

Cancel

**MX40** 

3

- 6) Confirm label assignment at MX40
- 7) If label already in use, Unassign from "Assigned Devices" section