



Pioneer Biomedical LLC

Supporting the Future, Repairing the Past.

1-888-644-3274

www.piobio.com

Removing E01 message on central with M2601A

1. Bottom right on Central station press ALL CONTROLS button
2. In the top right corner press the UNIT SETTINGS button
3. Select TELEMETRY FREQUENCY key in top right corner of screen.
4. Enter the following pass code M3150 (**CAPS LOCK OFF**).
5. The screen being displayed for you now is a list of all of the channels for that central. Each channel represents a sector on your screen.
6. Locate the sector you are working with by scrolling up and down with your keypad until you have the channel you are working with highlighted.
7. Put a known good battery in your transmitter
8. Now press the key labeled LEARN CODE.
9. Now press the nurse call button on your transmitter.
10. You should now have a signal on that sector.
11. Hook transmitter up to a patient simulator and admit the patient for that sector.

PIONEER BIOMEDICAL LLC

2815 Mercier St

Kansas City, MO 64108

(888) 644-3274